## Smoothdrive 1:5 Electric Contra Angle and Other Electric Attachments

### Caution: IMPORTANT SAFETY INFORMATION — READ BEFORE FIRST USE!

Electric dental motors have much more power than air motors or turbines and can generate excessive heat in worn, damaged or poorly maintained contra-angles. Serious patient burns can occur when contra-angles overheat during patient treatment. Burns may go undetected by dentist or patient until after tissue damage has occurred because the anesthetized patient cannot feel the tissue burning and the dentist is insulated from the heated contra angle by the handpiece housing.

#### TO AVOID PATIENT INJURY:

- Always use water spray when treating patients with the contra-angle.
- Electric contra-angles must be lubricated after each use. Maintain according to the instructions for use (see back of this card for how to access). Train office staff to properly clean and maintain the contra-angles. Implement a method for tracking that proper maintenance was performed for each instrument.
- Examine electric contra angles before EACH use for excessive heat, noise or vibration. Immediately stop using any suspect instrument and contact Lares Research at 1-888-333-8440, ext. 4 to arrange repair.
- Do not use worn or bent burs.
- To avoid overheating of the push-button, which could cause patient burns, it should not be pressed inadvertently while the instrument is rotating. Soft tissue (tongue, cheek, lips, gums, etc.) must be protected by deflection with a shield, retractor or a dental mirror. Never use the contra-angle as a tongue or cheek retractor.
- Do not use water to cool contra-angle after removal from autoclave as rapid contraction could damage product.

Questions? Call Lares technical services at 1-888-333-8440, ext. 2.

Lares Research  
295 Lockheed Avenue  
Chico, CA 95973  
USA

[www.laresdental.com](http://www.laresdental.com)

Item 72809 REV 11/23

**lares**<sup>®</sup>  
RESEARCH  
*Precision Performance*

## DOWNLOADING INSTRUCTIONS FOR USE

[www.laresdental.com/support](http://www.laresdental.com/support)

**En:** Please read the instructions before using this Lares product. You can download these instructions at [www.laresdental.com/support](http://www.laresdental.com/support) or order them by phone. For repairs, please contact a representative at **888-333-8440 ext. 2.**

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