

# Pastel™ Prophy Angles Instructions For Use

Single use item, discard after a single use on one patient. Contamination could occur if reused. **Warning:** This single-use device should only be used with a sterilized handpiece between patients. The device contains small components that can pose a choking hazard. Not made with natural rubber latex.

## Instructions for Use

- Attach the prophy angle to your handpiece and run up to 4000 RPM's (67 Hz).
- Maintain constant speed and contact, for as long as necessary to polish and clean all surfaces, including deep and stubborn stains. There is no need to lift, dab or dot on and off the teeth; Pastel will not generate any uncomfortable heat and will not stall while using firm pressure.
- Completely seat Pastel and operate with the drive shaft in alignment to the axis of the handpiece. DO NOT flex at an angle to the handpiece.


**REF** 11646

**Manufactured for:**

Lares Research  
295 Lockheed Avenue  
Chico, CA 95973 U.S.A.

All product names are trademarks of Lares Research.  
©2025 Lares Research, All rights reserved.

TPALA-INS / REV 1 - 02/2025

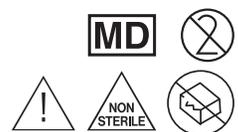

**LaRES**<sup>®</sup>  
RESEARCH  
Precision Performance